

## **SAMPLE CONSERVATION**

To obtain the best results possible, sample should be stored correctly.

Please refer to the following table for the storing condition for each hormone.

<b>Hormone</b>	<b>Storing at 2-8°C</b>	<b>Storing at -20°C</b>
Progesterone	Stable for 5 days	Stable
Estradiol	Stable for 5 days	Stable
Testosterone	Stable for 5 days	Stable
ACTH	Little stable	Stable for 4 months
Aldosterone	Stable for 4 days	Stable for 2 months
Cortisol	Stable for 5 days	Stable for 1 month
Corticosterone		Stable
TSH	Stable for 1 week	Stable for 2 months
T4	Stable for 5 days	Stable for 1 month
fT4	Stable for 5 days	Stable for 1 month
T3	Stable for 5 days	Stable for 1 month
fT3	Stable for 5 days	Stable for 1 month
PTH	Stable for 5 days	Stable for 1 month If stored at -80°C stable for 2 months
25-OH Vit. D	Stable for 24 hours	Stable
1, 25 (OH) <sub>2</sub> Vit. D	Stable for 24 hours	Stable